



CLINICAL ASSISTANCE GUIDE

ASPIRE SOCIALITE FOLDING WHEELCHAIR

Aspire Socialite Folding Wheelchair

The Aspire Socialite is an ultra-light weight transport chair perfect for short trips and outings with friends. The generous 115kg safe working load caters to a broad range of users. Padded and breathable support surfaces and a contoured back, provide optimum comfort and postural support. Offset uprights and cut away armrests provide a sleek and stylish modern look, as well as functionality, with improved table access and space for coats and warmer clothing.

AVAILABLE IN 2 MODELS:

Aspire Socialite - Attendant Propelled (AP): The Aspire Socialite AP model has 12-inch rear wheels.



Aspire Socialite - Self Propelled (SP): The Aspire Socialite SP model has 22-inch removable rear wheels.



Both models have four-wheels and a cross-folding aluminium frame, making them light and easy to store and transport. They also offer great manoeuvrability, stability, height adjustable footplates for a customised fit and are equipped with push handles with attendant operated brakes, swing-away and lift off legrests and padded armrests.

Both models are available in 2 colour options:

- Red frame with red and black upholstery
- Silver frame with silver and black upholstery

FEATURES

Lightweight frame design	One of the lightest transport chairs available, the Aspire Socialite is an ultra-light weight manual wheelchair, perfect for short trips and outings with family and friends
Contemporary design	Sleek European design with stylish modern colours
Easy transportation	Simple and intuitive folding back and seat make it easy to 'fold and go'
Compact storage	Removable leg rests and folding back for compact storage
Contoured seating	Padded and breathable seat and back, and back contouring provide optimum comfort and postural support
Cut-away armrests	Offset uprights create space for coats and warmer clothing and improve table access
Attendant operated hand brakes	With parking lock, for increased safety and ease of use
Swing away and removable legrests	Assist with stand-transfers, as well as lifting and stowing the wheelchair
Adjustable height footplates	With safety heel straps
Available in 2 models	<ul style="list-style-type: none">• Attendant Propelled with 12" rear wheels• Self-Propelled with 22" rear wheels
Available in 2 colours	<ul style="list-style-type: none">• Red frame with red and black upholstery• Silver frame with silver and black upholstery
Safe Working Load	Generous 115kg SWL caters to a broad range of users
Anterior Seat Height Adjustment	25mm of anterior seat height adjustability <ul style="list-style-type: none">• Anterior seat height - 475 - 500mm• Posterior seat height - 440 mm
Rear storage pocket	To store small essentials when out and about
Anti-tippers	Anti-tippers – available as an accessory WCA848270

IDEAL USER CONDITIONS

The Aspire Socialite Folding Wheelchair is an ultralight wheelchair designed to enable access for community activities such as appointments, shopping trips and social outings and would assist:

- Frail and older people
- Those requiring a transport wheelchair due to reduced mobility, safety or endurance
- Family and formal carers

All Socialite wheelchair models are designed for users under 115kg in weight who can be comfortably seated in a fixed 450mm (17.7") wide x 430mm (17") deep wheelchair seat. The open arm design accommodates different body shapes and extra seasonal clothing.

Considerations:

- The Socialite AP wheelchair is able to be propelled by an attendant with the push handles
- The Socialite SP wheelchair can be propelled by the user with the large rear, self-propelling wheels, or by an attendant with the push handles
- These wheelchairs are designed for indoor use and outdoors on dry, hard and level terrain
- The Socialite AP and Socialite SP wheelchairs are not designed for the occupant to be transported in a vehicle whilst seated in the wheelchair. If this is required, ask our sales consultants about other wheelchair options
- The Socialite wheelchairs are intended for short duration use, for wheelchair users who can transfer independently, or with supervision or minimal assistance only, and do not require additional support for pressure care management. They are not intended for use as an all-day chair. If this is required, please discuss with our Equipment Specialists or Complex Rehabilitation Team

CLIENT OUTCOMES

Assisted mobility within the home:

- Compact and lightweight wheelchair is suitable to use within the home to support those with limited balance, mobility or endurance
- Large rear wheels and the lightweight frame enable the user to self-propel or to be assisted to access areas of their home
- Easy, compact folding enables convenient storage when not in use

Assisted local community access:

- The folding wheelchair can be placed in the boot or back of a car for mobility support during community visits such as appointments, shopping trips or social outings

Supported socialisation, health and wellbeing:

- Wheelchairs provide increased mobility and independence for the user and their carer, allowing them to take part in everyday activities, despite having reduced mobility
- A wheelchair can allow the user to continue to enjoy their favourite activities, whether that's a walk around the local park, visiting the local shops, catching up with friends or heading to a gallery. A wheelchair gives the user a more active lifestyle that may result in a more positive outlook on life
- The wheelchair's design features such as contoured and padded seating, improves postural support and comfort when out and about

Carer Support:

- The lightweight and compact frame make it easier for formal or family carers to fold and lift the wheelchair in and out of the car, and for quick set up on arrival
- Attendant operated handbrakes with parking lock improve safety for the carer and wheelchair user

SPECIFICATIONS

Overall Length	Overall Width	Overall Height	Seat Depth	Seat Width	Anterior Seat Height	Posterior Seat Height	Backrest Height	Armrest Height	Safe Working Load	Rear Wheel	Folded DxWxH
mm/ inches	mm/ inches	mm/ inches	mm/ inches	mm/ inches	mm/inches	mm/inches	mm/ inches	mm/ inches	kg/stone	mm/ inches	mm/inches

ATTENDANT PROPELLED: MWS449800 Silver & MWS449810 Red

1030/41	595/24	900/36	430/17	450/18	475-500 /19-20	440/18	410/16	200/8	115/19	305/12	650x305x740 /26x13x30
---------	--------	--------	--------	--------	-------------------	--------	--------	-------	--------	--------	--------------------------

SELF PROPELLED: MWS449820 Silver & MWS449830 Red

1030/41	675/27	900/36	430/17	450/18	475-500 /19-20	440/18	410/16	200/8	115/19	559/22	650x380x740 /26x15x30
---------	--------	--------	--------	--------	-------------------	--------	--------	-------	--------	--------	--------------------------

MINIMUM CHAIR WEIGHTS



Attendant Propelled(AP)



Self Propelled(SP)

COMPLIANCE

AS/NZS 3695.1:2011

WARRANTY

5 YEARS WARRANTY
Frame

1 YEAR WARRANTY
Fabric & Components

Minimum weight is chair weight minus the legrests for the AP model and for the SP model, minus the wheels and legrests.

MAINTENANCE

Weekly	<ul style="list-style-type: none"> • Check brakes are functioning properly
Periodically	<ul style="list-style-type: none"> • Check for signs of wear on front castors • Check for signs of wear on rear tyres
Yearly	<ul style="list-style-type: none"> • Service to be completed by authorised Aspire service agent
Serviceable Components	<ul style="list-style-type: none"> • Wheels & Tyres • Forks • Axles • Armrest receivers • Seat sling • Footrest mounts • Footplates • Armrest Pads

Care for Your Chair

- The seat and backrest upholstery can be wiped with a wet sponge and a mild detergent. All other components can be cleaned with a damp cloth.
- Check tyre condition at least twice per month.
- Check that rear wheel spokes are not loose. Loose wheel spokes can reduce the strength of the rims and must be safely adjusted by a qualified technician.
- Always keep rear wheel axles clean and free from dirt and hair build up.
- Brakes should be checked regularly.

Should you have any doubts about the correct functioning of the wheelchair or of any of its components, please contact AIDACARE on 1300 133 120 or your local Aspire distributor.

The tool kit provided will allow basic adjustments to the wheelchair. For further information on repairs and service contact AIDACARE on 1300 133 120.

AIDACARE KNOWLEDGE HUB

Aidacare Website:

- [ASPIRE SOCIALITE FOLDING WHEELCHAIR - ATTENDANT PROPELLED | AIDACARE](#)
- [ASPIRE SOCIALITE FOLDING WHEELCHAIR - SELF PROPELLED | AIDACARE](#)

Aidacare Flyer:

- [ASPIRE-SOCIALITE-WHEELCHAIR-FLYER.PDF \(AIDACARE.COM.AU\)](#)

Aidacare user manual:

- [ASPIRE-SOCIALITE-WHEELCHAIR-USER-MANUAL.PDF \(AIDACARE.COM.AU\)](#)

Video Library:

- [Introducing the Aspire Socialite Wheelchair](#)
- [Aspire Wheelchair Range Overview](#)